



Characteristic	Possible values
Usage	Operation 24/7 (dedicated machine recommended, hibernation and network card sleep mode disabled)
OS	Windows Server 2016® x64 Standard Edition with Desktop Experience (graphical interface) Windows Server 2012® R2 x64 Windows 10® x86 and x64 Pro edition
Processor	4-core - 64 bits - 2 Ghz
RAM	4 Gbytes
Disk type	Mechanical hard disk > 10 000 rpm or SSD
Typical recommended disk space	Space necessary for the OS + : - licence 1200 reports / day = 50 Gbytes - licence 2400 reports / day = 75 Gbytes - licence 4000 reports / day = 110 Gbytes - licence 6500 reports / day = 170 Gbytes - licence 10 000 reports / day = 250 Gbytes - licence 15 000 reports / day = 440 Gbytes
.NET Framework	Version 4.5.1 minimum
DB	No database engine installed
Setup configuration / FireWall - Network	<ul style="list-style-type: none"> • Fixed IP mandatory • Authorise incoming TCP protocol connections on the port(s) used to communicate with the LIS (by default port 7200 if only one LIS connection) • Authorise incoming TCP and UDP protocol connections on the ports used to communicate with the Valab® services of the Valab® Server and with the Valab® clients (by default 8080, 8089, 8090 and 2110)
Access	Local administrator account dedicated to the VALAB company Telemaintenance access (VPN + RDP, remote control software,...) Internet access to the "valab.com" domain
Backup / Security	Policy and implementation to be determined by the IS Manager of the medical laboratory